HYPURE™ OIL LIGHT

KITAZATO®

EN HYPURE™ OIL LIGHT

INTENDED USE

Hypure™ Oil is intended to be used as an overlay for culture of embryos, oocytes and sperm in ART and micromanipulation procedures.

COMPOSITION:

Pharmaceutical grade quality paraffin oil.

GENERAL INFORMATION:

Hypure™ Oil is pre-washed, ready to use mineral oil for use in IVF, ICSI and related artificial reproductive techniques.

The oil allows small volumes of medium to be used as it prevents the evaporation of media and maintain stable osmolality and pH in the culture medium.

INSTRUCTIONS FOR USE:

CAUTION: The detailed usage of Hypure™ Oil is determined based on each laboratory's procedure and protocols that are optimized for individual medical purposes.

Preparation :

- For culture, pre-equilibrate Hypure™ Oil as necessary, and the culture medium at 37°C in a CO₂ incubator for several hours (or overnight),
- For ICSI, warm Hypure™ Oil and HEPES buffered medium at 37°C.

Instructions for use :

1. Put culture medium on a dish or place micro drop of culture medium on a dish.
2. Gently pipette equilibrated Hypure™ Oil over the medium until they are completely covered.

NOTE : Washing of oil for embryo culture is not required.

QUALITY CONTROL SPECIFICATIONS:

The following tests were performed for each lot of this product:

- Sterility: SAL 10⁻³
- Endotoxin: < 0.1 EU/ml
- Mouse embryo assay: ≥ 80% blastocysts after 120 hours in culture
- Density: 0.83-0.86 g/ml
- Viscosity: < 30 cP at 30°C
- Peroxide Value (POV): <0.1mEq/kg

Kitazato Corporation
100-10 Yanagishima, Fuji, Shizuoka 416-0932 Japan
TEL: +81-545-65-7122 FAX: +81-545-65-7128

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