Instructions For Use Kitazato OPU Needle

EN : General Information

Refer to the label on the product or package before use.

Symbols on the Label

	Do not reuse		Batch code
	Use by		Re-order number
	Consult Instructions for use		Manufacturer
	Sterilized using ethylene oxide gas		EC representative
	Sterilized using irradiation		Do not re-sterilize
	Do not use if package is damaged		

Intended use

The Kitazato OPU Needle is intended to puncture into human body and pick up follicular fluid or oocyte from ovarian follicle.

Contraindications

- No reuse. It may cause infection when reuse.
- No re-sterilization

Instructions for use

- (1) Apply ultrasound probe to the puncture region and check the condition of the follicle. For transvaginal puncture, make sure of the adjustment for the transvaginal ultrasound probe guide and the gauge of the needle.
- (2) Connect the needle hub or the female connector hookup to the syringe or the suction pump and make sure the direction to the follicle looking at the monitor of ultrasound.
- (3) Insert the needle tip to the follicle and aspirate the follicular fluid slowly.
- (4) For puncturing another follicle in the same ovary, keep the needle tip in the ovary. It is recommended to do flushing with the 10mL syringe accompanying the Double Lumen of OPU Needle. Conduct the flushing slowly.
- (5) Remove the needle after aspirating the follicular fluid.
- (6) Lightly press the puncture region with gauze. Apply an adhesive plaster to the puncture region if bleeding can not be observed. Check the condition of the patient for 15 to 30 minutes.
- (7) Immediately do speculum or store for the aspirated tissue and cell.

Cautions for use

- (1) Confirm the specification (gauge and length) of the needle before opening it.
- (2) Confirm adaptability with the other medical equipment to use together beforehand.
- (3) Pay enough attention to the needle tip and do not touch it until it is used for puncture. An efficacy of the puncture may deteriorate.
- (4) Confirm a route of the puncture enough before puncturing the needle.
- (5) When the dogging in the needle or the connecting tube is confirmed on aspiration of tissues, remove the dogging from it by pressurizing with the cannula attached to the product. However, avoid the enforced handling, and, depending on the situation, change a needle newly.
- (6) Read an Instructions For Use before use by all means.
- (7) Do not use the product that it was past an expiration date.
- (8) If the product has abnormalities that its package has been broken or it has been damaged before opening it, do not use it.
- (9) Use the product by operator that has seasoned knowledge about procedure for the product.
- (10) Do not use the product except the collection of the tissues and the cells.
- (11) Use the product aseptically. Carry out enough precautionary measures for bacteria.
- (12) Confirm leakage through solution such as saline before use. Do not use the product when air bubble is confirmed by through solution such as saline.
- (13) Do not recap after use.
- (14) Pay enough attention to a false puncture.
- (15) Throw the product away promptly after having used it.
- (16) Do not refurbish.
- (17) Immediately use after opening the package.
- (18) Store away from high temperature, high humidity, the water wet.

Warranty

The manufacturer is not liable for any damage resulting from misuse or incorrect operation or failure to follow the cautions or instructions contained in this Instructions For Use or the instruction manual for any compatible medical equipment.

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