
Vitrification Cryotop Method for Oocyte and Embryo Thawing Media

CE 2797

Code:VT802

TS Warming
(+42000°C/min)
1min, 37°C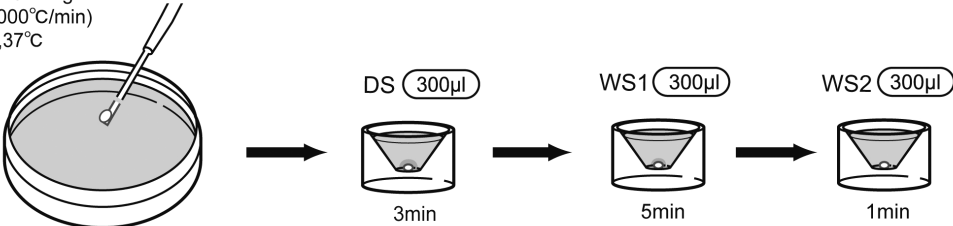

EN: Intended use

This product is to be used for thawing of oocytes(MII) and embryos.

Thawing Media

- No. 1 (Color code: Red) Thawing Solution (TS): 2x4.0 ml vials
- No. 2 (Color Code: Yellow) Diluent Solution (DS): 1x4.0 ml vial
- No. 3 (Color Code: White) Washing Solution (WS): 1x4.0 ml vial

Caution: Before use, check the specifications of the container and labeling (number, color, lot, and volume). If you notice anything unusual regarding the items mentioned above, do not use the product and please contact the distributor.

Recommended equipment

- PetriDish: 35 mm for TS
- ReproPlate: with 6 wells
- Caution:** This product is sterilized. Please handle the product under sterilization in a clean bench.

Instructions for use (IFU)

Preparation
Warm TS vial with a cap and a Petri Dish in an incubator to 37°C. Pour the full content of TS into the Petri Dish.
Drop 300 µl each of DS, WS1 and WS2 on the Repro Plate with a micropipette.
Caution: Use a sterilized micropipette as a handling tool, with a suitable internal diameter for oocytes or embryos. The recommended internal diameters are as follows: 180 µm for oocytes(MII), 180 µm for pronuclear stage embryos, and 250 µm for cleavage or blastocyst stage embryos.

Thawing

1. Quickly immerse the Cryotop strip completely into the TS. Leave it for 1 minute.
2. Aspirate the oocyte(MII)/embryo with the micropipette and gently place it on the BOTTOM of the DS. Leave it for 3 minutes.
3. Aspirate the oocyte(MII)/embryo with the micropipette and gently place it on the BOTTOM of the WS1. Leave it for 5 minutes.
4. Aspirate the oocyte(MII)/embryo with the micropipette, gently place it on the SURFACE of the WS2. After the oocyte(MII)/embryo drops to the bottom of the WS2, repeat this process one more time.
5. Transfer the oocyte(MII)/embryo to a culture dish containing the appropriate culture medium. Incubate the oocyte(MII)/embryo in a 37°C incubator to complete recovery.
Caution 1: It is recommended to incubate oocytes(MII) and embryos for 2 hours.
Caution 2: Wash the embryo and incubate embryo for recovery in appropriate media to avoid WS2 to be transferred in patient body.

Quality control specification

The following tests were performed for each lot of this product:

- Sterility by the current USP Sterility Test <71> (Solutions)
- Endotoxin by LAL methodology
- Mouse Embryo Assay (OneCell)
- pH (Ph.Eur., USP)
- Osmolality (Ph.Eur., USP)

Storage instructions and stability

Solutions: Store the vials at 2 to 8 °C
This product is stable until the expiry date shown in the labeling on the vial.

Composition

- HEPES within Basic Culture Medium
- Trehalose
- Hydroxypropyl Cellulose

Contraindication

- Do not re-sterilize
- Do not re-use
- Do not use solution that shows cloudiness or has turned yellow.
- Do not use the product if you notice anything unusual regarding the specifications on the label (number, color, name, volume).
- Sterile if the package or container is unopened or undamaged. Do not use if the package or container are opened or damaged.
- Upon delivery media must be stored in original unopened container and refrigerated at 2-8 °C.
- Do not use and please discard if the media were not stored under refrigeration (2 to 8 °C).
- Do not use the product if past the expiration date.

Warning

- Read the instructions for use prior to use.
- This product is intended to be used by medical specialists trained in fertility treatment.
- Aseptic technique should be used.
- Use sterilized equipment and materials only.
- Decontaminate the workroom for aseptic procedures.
- Follow procedures in an controlled environmental room.
- Observe all federal, state and local environmental regulations when discarding the product.
- Wear sterilized gloves, hat, white coat to prevent contamination
- Use timer during operation. Prepare another timer for backup in case the battery runs out.
- In case of eye or skin contact with Vitrification/Thawing media, immediately flush eyes/skin with water.
- Operation must be conducted by two people and perform a double check.
- Request a qualified waste disposal contractor to dispose the products as residual and non-reusable solutions.
- Dispose the contaminated containers and packages in the same way as the containers with the product inside.
- The user shall be responsible for any problems caused by non-conformity to the present IFU.

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Kitazato Corporation